

# P90




Volume of water: Ensure the cooling water reach the end of the tip

Functional Description: Used for gingival large stones and dental plaque



Applicable area: All teeth buccal and lingual site of dental implant or prosthesis.

Operation method: The tip is parallel to the long axis of the tooth, and ensure the 2 mm right or left tip middle site adapt the tooth surface. Move the tip front and back.

Warn: Please ensure that there is water coming out from the handpiece to avoid damage to the tip

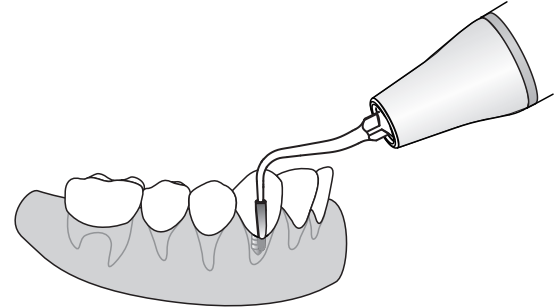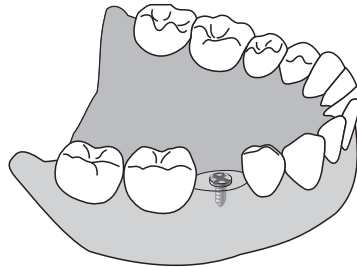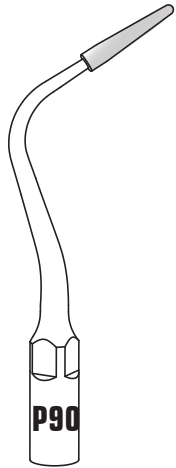

**MODE: P**  
**POWER**

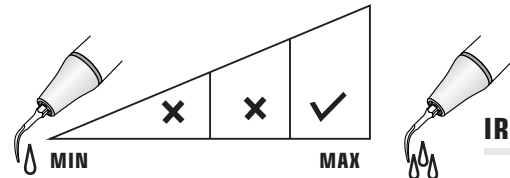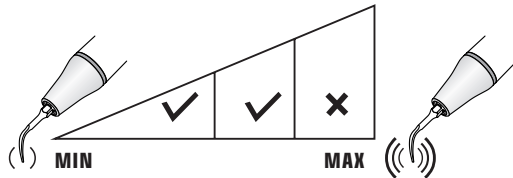

**IRRIGATION**