

P95

Function: Cleaning and maintenance of implant

Size: Plastic head is of 19mm length

Features: The tip is of a slender hook shape, making it easier for subgingival cleaning and maintenance.

Use:

a. Used in P mode of Periodontal Treatment Device. Recommended power level: level 1~6.


b. Used in P mode of U600. Recommended power level: level 1~3

c. Used in P mode of U6. Recommended power level: level 1~6.



Note:

(1) The plastic part is made of PEEK material which can be disinfected under high temperature and high pressure, the higher the power, the easier it is to wear;

(2) Avoid the tip stuck in the gap of the tooth during operation to avoid breaking the tip.

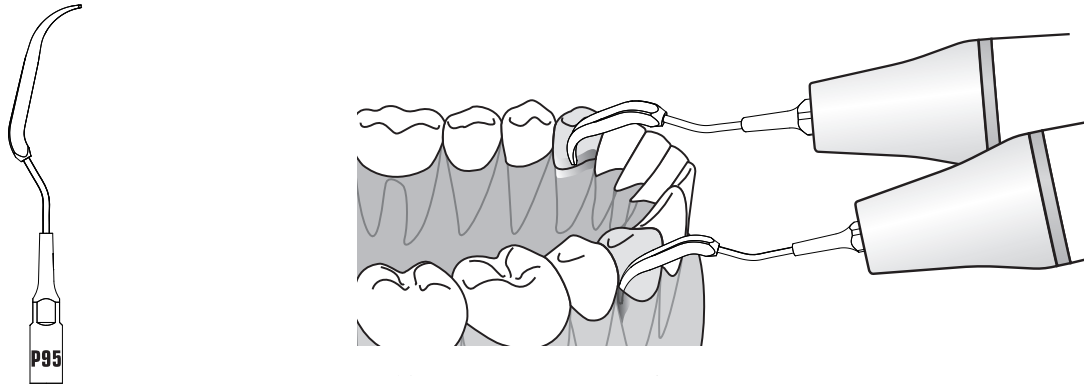

MODE: P
POWER

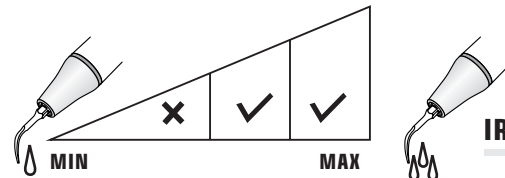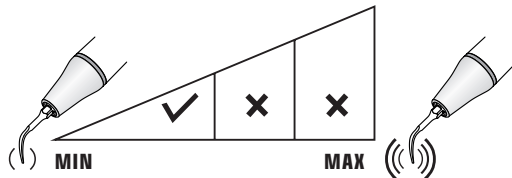

IRRIGATION